

# Technipro

**Marketing Pty Ltd**

ABN 54 002 730 421

## **INSTRUCTIONS FOR USE AND CARE OF PARI TURBO BOY N AND TURBO BOY SX**

Only medication prescribed or recommended by the clinician or doctor should be used.

Make sure side air vents are unobstructed. Do not cover the compressor while in operation.

Ensure that the tubing is not kinked, this restricts airflow. Replace kinked tubing.

Before cleaning always switch off compressor and disconnect the power cord from wall socket.

Do not spray cleaning liquids into the side air vents, moisture is likely to damage electrical or other compressor parts and cause malfunction.

Under normal operation the filter in the front panel needs to be replaced every 500 hours or at least once a year. Inspect every 10<sup>th</sup> inhalation or so. If soiled or blocked (grey or brown staining) replace filter.

Operate compressor on clean hard surface i.e. table or bench top.

Do not operate compressor by placing it on the carpet.

The Pari Compressor has a warranty period of 3 Years from date of purchase. The warranty is void if repairs are undertaken by unauthorised persons.

Should a warranty repair be required on the compressor (within the 3 year period) Technipro Marketing is to be notified. All repairs are carried out in Sydney by an authorised Pari technician.

The compressor when not in use should be stored in a clean and dry environment. Ideally in the original carton when purchased.

Office Address: 10 / 13 Berry Street CLYDE NSW 2142

**Postal Address: PO Box 6390, Parramatta Business Centre NSW 2150**

**Tel: (02) 9897 5899 Fax: (02) 9897 5799**

Email: [service@technipro.com.au](mailto:service@technipro.com.au)